# INHEALTH

TECHNOLOGIES®

## Blom-Singer® Manometer with Manometer Adaptor

### INSTRUCTIONS FOR USE

**Blom-Singer®**

voice restoration systems

37469-01H Effective January 2020 / 2020 年 1 月生效 / 2020年1月発効 / Në fuqi nga janari 2020 / 2020 اعتبارًا من شهر يناير / В сила от януари 2020 г. / Stupa na snagu u siječnju 2020. / V platnosti od ledna 2020 / Gældende fra januar 2020 / Met ingang van januari 2020 / Kehtib alates jaanuarist 2020 / Voimassa tammikuusta 2020 alkaen / Date d'entrée en vigueur : janvier 2020 / Gültig ab Januar 2020 / Σε ισχύ από το Ιανουάριο του 2020 / 2020 בתוקף החל מינואר / Hatályos 2020. januártól / Gildir frá janúar 2020 / Valido da gennajo 2020 / 2020년 1월 발효 / Spēkā no 2020. gada janvāra / Įsigalioja 2020 m. sausio mėn. / Gjelder fra januar 2020 / Wchodzi w życie w styczniu 2020 r. / Vigente em janeiro de 2020 / Cu intrare în vigoare din ianuarie 2020 / Версия от января 2020 г. / Važi od januara 2020. / Účinné od januára 2020 / Efectivo a partir de enero de 2020 / Gäller från januari 2020 / Geçerlilik Tarihi: Ocak 2020

Blom-Singer® and InHealth Technologies® are registered trademarks of Freudenberg Medical, LLC.

Patent(s): [www.inhealthpatents.com](http://www.inhealthpatents.com)



Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.

## INSTRUCTIONS FOR USE

使用说明 / 使用說明書 / Udhëzime për përdorim / إرشادات الاستخدام / Инструкции за употреба / Upute za korištenje / Návod k použití / Brugsanvisning / Gebruiksaanwijzing / Kasutusjuhend / Käyttöohjeet / Instructions d'utilisation / Gebrauchsanweisung / Οδηγίες χρήσης / הוראות שימוש / Használati utasítás / Notkunarleiðbeiningar / Istruzioni per l'uso / 사용 지침 / Lietošanas norādījumi / Naudojimo instrukcijos / Bruksanvisning / Instrukcja użytkowania / Instruções de uso / Instrucțiuni de utilizare / Инструкции по применению / Uputstvo za upotrebu / Návod na použitie / Instrucciones de uso / Bruksanvisningar / Kullanım Talimatlar

## DIAGRAMS

插图 / 図 / Diagramet / الرسم التوضيحي / Схеми / Dijagrami / Schémata / Diagrammer /  
 Afbeeldingen / Diagrammid / Kaaviot / Diagrammes / Abbildungen / Διαγράμματα / תרשימים /  
 Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Diagramos / Diagrammer /  
 Schematy / Diagramas / Diagreame / Схеми / Crteži / Grafické znázornenia / Diagramas /  
 Diagram / Diyagramlar

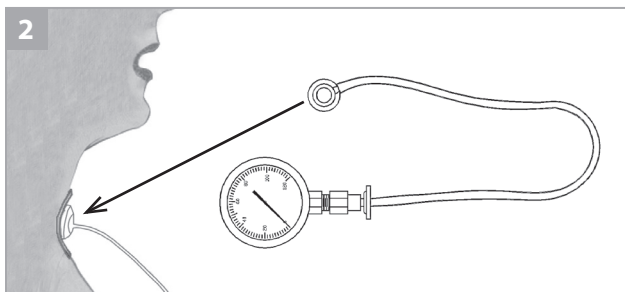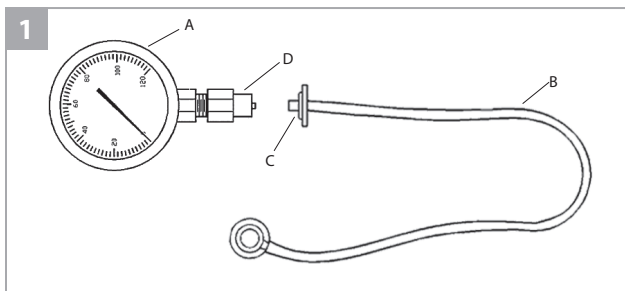## TABLE OF CONTENTS

目録 / 目次 / Tabela e përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /  
 Indholdsfortegnelse / Inhoud / Sisukord / Sisällysluettelo / Table des matières /  
 Inhaltsverzeichnis / Πίνακας περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /  
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis treści / Índice / Cuprins /  
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

|    |  |    |    |  |    |
|----|--|----|----|--|----|
| EN | Manometer With Manometer Adaptor           | 5  | HU | Manométer manométeradapterrel                        | 37 |
| ZH | 附带压力表适配器的压力表                               | 7  | IS | Þrýstingsmælir ásamt millistykki fyrir þrýstingsmæli | 39 |
| JA | マンメーターアダプター付きマンメーター                        | 8  | IT | Manometro con adattatore per manometro               | 41 |
| SQ | Manometër me përshatës manometri           | 11 | KO | 압력계 어댑터를 포함하는 압력계                                    | 43 |
| 13 | جهاز مقياس الضغط المزود بموانم مقياس الضغط | AR | LV | Manometrs ar manometra adapteru                      | 45 |
| BG | Манометър с адаптер за манометър           | 15 | LT | Manometras su manometro adapteriu                    | 47 |
| HR | Manometar s manometarskim prilagodnikom    | 17 | NO | Manometer med manometeradapter                       | 49 |
| CS | Manometr s adaptérem manometru             | 19 | PL | Manometr ze złączem manometru                        | 51 |
| DA | Manometer med manometeradapter             | 21 | PT | Manómetro com Adaptador de manómetro                 | 53 |
| NL | Manometer met manometeradapter             | 23 | RO | Manometru cu adaptor pentru manometru                | 55 |
| ET | Manomeeter manomeetri adapteriga           | 25 | RU | Манометр с адаптером манометра                       | 57 |
| FI | Manometri ja manometrisovitin              | 27 | SR | Manometar sa adapterom za manometar                  | 59 |
| FR | Manomètre avec adaptateur pour manomètre   | 29 | SK | Manometer s adaptérom manometra                      | 61 |
| DE | Manometer mit Manometeradapter             | 31 | ES | Manómetro con adaptador para el manómetro            | 63 |
| EL | Μανόμετρο με προσαρμογέα μανόμετρου        | 33 | SV | Manometer med manometeradapter                       | 65 |
| 35 | מד-לחץ, עם מתאם למד-לחץ (מנומטר)           | HE | TR | Manometre ve Manometre Adaptörü                      | 67 |

## ENGLISH

**BLOM-SINGER® MANOMETER WITH MANOMETER ADAPTOR****PRODUCT DESCRIPTION**

The manometer (with manometer adaptor) is supplied nonsterile in a kit containing one manometer, one manometer adaptor and one Blom-Singer Adhesive Housing. Replacement manometer adaptors and housings are available separately. The manometer is reusable between patients. The manometer adaptor and housing are disposable and intended for single use only.

| Manometer with Adaptor             | Manometer | Adaptor | AccuFit Housing |
|------------------------------------|-----------|---------|-----------------|
| Manometer Kit                      | •         | •       | •               |
| Replacement Manometer Adaptor Pack |           | •       | •               |

**INDICATIONS** (Reasons to prescribe the device or procedure)

The manometer (diagram 1A) (with manometer adaptor) is for use by qualified, trained clinicians as an aid to teach laryngectomized patients how to self-regulate intratracheal pressure during tracheoesophageal speech.

**CONTRAINDICATIONS** (Reasons that make it inadvisable to prescribe the device or procedure)

The manometer (with manometer adaptor) is only indicated for laryngectomized patients who speak using a tracheoesophageal voice prosthesis.

**WARNINGS AND PRECAUTIONS**

If there are tears, cracks, or structural damage, do not continue to use the product. The manometer adaptor and housing are disposable and intended for single use only; reuse may result in cross contamination. The manometer is pre-calibrated at the manufacturing facility. Tampering with the instrument (e.g., unscrewing the device) may result in malfunction, inaccurate pressure readings, and/or leaks of the gauge. Tampering with the connection fittings on the manometer adaptor may result in inaccurate pressure readings.

**INSTRUCTIONS FOR USE**

Please refer to the diagrams located at the front of this instruction manual.

The clinician's hands should be clean and dry to help avoid introducing contaminants resulting in patient infection. The included Blom-Singer adhesive housing must first be attached to the skin surrounding the patient's tracheostoma. Follow the supplied Instructions for Use manual for instructions on application of adhesive housing. The manometer adaptor (diagram 1B) should be connected to the manometer before attaching the manometer adaptor to the tracheostoma adhesive housing.

Manometer Adaptor Attachment

1. Attach the manometer adaptor to the manometer by inserting its female luer lock connection (diagram 1C) into the male luer lock connection (diagram 1D) of the manometer. Twist the two components together until securely connected.
2. Attach the manometer adaptor to the patient by inserting the tracheostoma connector end to the tracheostoma adhesive housing (diagram 2).

### Intratracheal Speech Pressure Assessment Procedure

The following procedure is provided by Eric D. Blom, Ph.D.

With the manometer adaptor inserted into the attached tracheostoma adhesive housing, instruct the patient to take a breath, occlude the adaptor with a finger, and count to 15 with normal conversational speech effort. Optimal levels are designated by color markings on the manometer gauge (see chart below). Higher levels may contribute to suboptimal voice quality, increased fatigue with voicing, and shorter duration of the tracheostoma adhesive housing seal.

| Pressure Ranges (cm of H <sub>2</sub> O) |                   |
|--|-------------------|
| 10 — 25 cm H <sub>2</sub> O              | Excellent — Green |
| 26 — 40 cm H <sub>2</sub> O              | Good — Blue       |
| 41 — 120 cm H <sub>2</sub> O             | High              |

For some patients, observing the manometer as they vary their expiratory effort from optimal levels (usually registering 10 – 15 cm H<sub>2</sub>O) to high (often measuring 90 cm H<sub>2</sub>O or more) gives them feedback to monitor their expiratory effort when voicing.

### Manometer Care

The manometer device is reusable and its external components may be wiped with an alcohol wipe or other similar wipe as needed.

### **SPECIAL STORAGE AND/OR HANDLING CONDITIONS**

There are no special storage and/or handling conditions for this device.

### **DISPOSAL INSTRUCTIONS**

This device is not biodegradable and can be contaminated when used. Carefully dispose of the device per local guidelines.

### **ORDERING INFORMATION**

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: [order@inhealth.com](mailto:order@inhealth.com) ORDER ONLINE: [www.inhealth.com](http://www.inhealth.com) POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service

### INTERNATIONAL

Please contact our customer service department for a distributor referral.

### **PRODUCT COMPLAINTS/EU SERIOUS INCIDENTS**

If you are dissatisfied with the device or have any questions, please contact

**[productcomplaints@inhealth.com](mailto:productcomplaints@inhealth.com)**.

Telephone: +1-800-477-5969

Fax: +1-888-371-1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

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**SYMBOLS GLOSSARY (USA only)**

| <b>SYMBOL</b>  | <b>STANDARD No.</b>      | <b>SYMBOL TITLE &amp; REF No.</b>                          | <b>MEANING OF SYMBOL</b>   |
|--|--------------------------|--|--|
|    | EN ISO 15223-1:2016      | Manufacturer; 5.1.1  | Indicates the medical device manufacturer, as defined in EU Directives or Regulations  |
|    | 93/42/EEC<br>EU 2017/745 | CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V       | CE marking of conformity   |
|    | EN ISO 15223-1:2016      | Authorized representative in the European Community; 5.1.2 | Indicates the authorized representative in the European Community  |
|    | EN ISO 15223-1:2016      | Batch code; 5.1.6  | Indicates the manufacturer's catalogue number so that the medical device can be identified   |
|    | EN ISO 15223-1:2016      | Batch code; 5.1.5  | Indicates the manufacturer's batch code so that the batch or lot can be identified   |
|   | EN ISO 15223-1:2016      | Use-by date; 5.1.4   | Indicates the date after which the medical device is not to be used  |
|    | EN ISO 15223-1:2016      | Caution; 5.4.4   | Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself |
|  | EN ISO 15223-1:2016      | Do not use if package is damaged; 5.2.8                    | Indicates a medical device that should not be used if the packaged has been damaged or opened  |
|  | EN ISO 15223-1:2016      | Consult instructions for use; 5.4.3                        | Indicates the need for the user to consult the instructions for use  |
|  | EN ISO 15223-1:2016      | Do not re-use; 5.4.2                                       | Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure  |

# INHEALTH

TECHNOLOGIES®

---

## UNITED STATES

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll-Free: 800.477.5969  
Toll-Free Fax: 888.371.1530

## INTERNATIONAL

Tel: +1.805.684.9337  
Fax: +1.805.684.8594  
E-mail: [order@inhealth.com](mailto:order@inhealth.com)

## ORDER ONLINE

[www.inhealth.com](http://www.inhealth.com)

**Freudenberg Medical, LLC**  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands